

First View फस्ट व्यू

Rapid test for Ferritin




For Qualitative Detection of Ferritin Whole Blood Rapid Test

- Whole Blood, Serum or plasma samples may be used for testing
- Results available in only 5 minutes
- Easy to use
- Results can be read visually
- No need for an analyser
- Cost effective method for assisting in the diagnosis of iron deficiency anaemia




READING TEST RESULTS

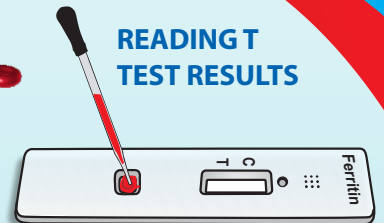

dispense one drop (25 μ L) of serum or plasma into sample well. If the whole blood is used, dispense two drops (50 μ L) into the sample well.


Wait for the whole blood sample to be completely absorbed before adding diluent.

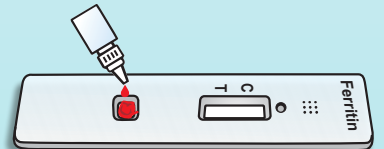


Add exactly 4 drops of diluent (150 μ L) in the sample well.



Result

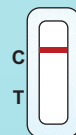

Positive

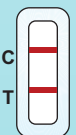

Negative