

First View फर्स्ट व्यू

Rapid test for Ovulation

Luteinizing Hormone (LH)




For Qualitative Detection of Luteinizing Hormone (LH) Urine Rapid Test

- Rapid Urine test for LH levels and find 2 most fertile days
- Results in 5 minutes
- Detects the LH hormone in urine at the FDA standard level of 25 mIU/mL
- Detects the hCG hormone in urine at the FDA standard level of 25 mIU/mL
- Easy to use
- High accuracy and reliability




READING TEST RESULTS

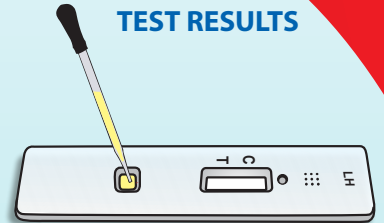


Collect urine sample in urine tray. Add 2-3 drops of urine into the sample well with the help of dropper.




Result

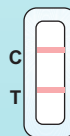

Positive
Your fertile days have started.

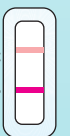

Negative
Daily testing should be continued.

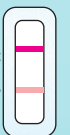

Invalid
Please test with another device.

