

First View फर्स्ट व्यू


Rapid test for Troponin-I

For Qualitative Detection of Troponin-I Whole Blood Rapid Test


- Excellent Screening Professional for STAT Diagnosis of Acute Myocardial Infarction (AMI)
- Run with WHOLE BLOOD or PLASMA or SERUM
- Highly sensitive assay with minimum detection level of 0.3 ng/ml
- Rapid test result in just 15 minutes
- Semi-quantitative point-of-care Test for Troponin I (cTnI)



READING TEST RESULTS

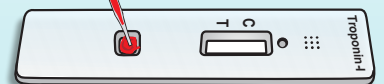


Add 2 drops (40µL) Whole Blood or 2 drops (40µL) of serum or plasma in sample well "S" using the dropper provided.

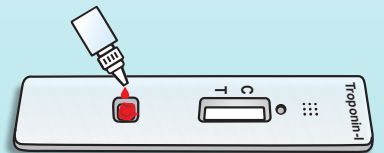


Add 2 drop of the Assay buffer



Result

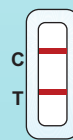

Positive

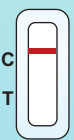

Negative



Invalid